CYTOLA[®] Ultrasound Gel

Ingredients: Deionised Water, Carbomer, Triethanolamine, Monopropylene glycol, Preservative.

EN PRODUCT DESCRIPTION

Non-sterile contact medium for ultrasound transmission

APPLICATION

► Apply enough gel on the transducer (probe). ► Follow the instructions of transducer (probe) manufacturer's instructions instead. ► It is recommended to use with a cover for the ultrasound probe / transducer.

OPERATION PRINCIPLE

In all ultrasonography and doppler applications, it pushes the air between the skin and the probe, providing better penetration of sonography waves into the device examination area and reducing losses in reflected waves.

INTENDED USE

Water-based non-sterile gel used as contact media in non-invasive, low risk medical procedures (diagnostic or therapeutic) where a viscous gel is required.

CONTRAINDICATIONS

► Wounds. ► Natural cavitations. ► Fresh surgical trace. ► Not suitable for; sterile, invasive, surgical, intraoperative, intracavitary and biopsy operation.

WARNING

► For medical professional use only ► Do not swallow. ► Do not keep the lids, covers, caps, flaps open. ► Skin disinfectant may be applicable onto examination area before and after the application. ► Stop the procedure in case of hypersensitivity. ► Clean the transducer with clean wipe and disinfect it acc.to manufacturer recommendations. ► A disposable bottle or bag is used for the possible infected patient. ► The bottle should not be left with the patient for a long time without the user.

CLAIMS: Non-toxic, Not-sensitive, Non-irritating, Non-staining, No formaldehyde, Paraben free, Salt free, Dermatologically tested, Hypo-allergenic, PVC free, Water soluble, Compatible with PE/PU/ Polyisoprene/Latex covers for ultrasound probes/transducers

► Follow the portioning instructions (5000 ml).

USE OF CONTAINER AND BOTTLES

5000 ML

► When filling the bottle, check the expiry date of the container. ► Do not touch the opening of the bottle with bare hand. ► Bottles are filled without touching gel ► Use up the gel from the bottle within 7 days, after which any remaining gel must be discarded and refilled again ► Disinfect the empty bottle.

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Cytolax is a Registered Trademark of Sharon Services (UK) Ltd., Glasgow, G32 8RF. sales@cytolax.co.uk
Sharon Services Europe B.V., Industrierweg 13-B, 1566 JN Assendelft, Netherlands. sales@sharon-services.eu.com