KONIX
Steril gel
Ultrasan
20 ml e

ULTRASONİK İLETKEN JEL

İÇERİK: Deionized water, Carbomer, Sodium-Hydroxide, MonoPropylene Glycol, Phenoxyethanol, Ethylhexylglycerin

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UK REP: Sensus Group Ltd
The Bank, Teddington
Farm Spellbrook, Herts
CM23 4BD United Kingdom

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KONIX
Sterile gel
Ultrasound
20 ml e

ULTRASONIC TRANSMISSION GEL

INGREDIENTS: Deionized water, Carbomer, Sodium-hydroxide, MonoPropylene Glycol, Phenoxyethanol, Ethylhexylglycerin

PRODUCT DESCRIPTION: Sterile contact medium for ultrasound transmission

WARNING:
·For medical professional use only
·Do not reuse
·Stop the procedure in case of hypersensitivity.
·Sterile technique is used to avoid contamination.
·Re-use of the device could result in cross-contamination and infection risk or adverse reaction for the patient.

INTENDED USE:
Disposable sterile gel as contact media for invasive medical procedures (diagnostic or therapeutic) where a viscous gel is required.

OPERATION PRINCIPLE:
In all ultrasonography and doppler applications, it pushes the air between the skin and the probe, providing better penetration of sonography waves into the device examination area and reducing losses in reflected waves.

APPLICATION:
·Apply enough gel on the transducer (probe).
·Follow the instructions of transducer (probe) manufacturer's instructions instead.
·It is recommended to use with a sterile cover for the ultrasound probe /transducer.

NOTES:
·Non-toxic ·Not-sensitive ·Non-irritating ·Non-staining ·No formaldehyde ·Paraben free ·Salt free ·Dermatologically tested ·Hypo-allergenic ·PVC free ·Water soluble ·Compatible with PE/PU/ Polyisoprene/Latex covers for ultrasound probes/transducers

STERILE|R MD

CYTOLAX®
Imported & Distributed in the UK by:
Cytolax, Shilton Services (UK) Ltd.,
Glasgow G32 8RF. sales@cytolax.co.uk

CE
2292

REF: SUG-0001 VERS: Y2064102.0001